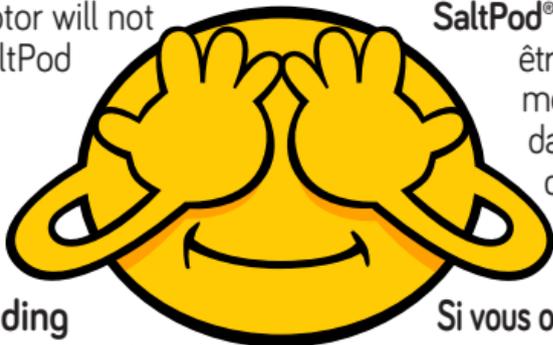
The Naväge® Nose Cleaner only works with genuine Naväge SaltPod® capsules. The power button cannot be pushed in and the motor will not turn on unless a new, unused SaltPod has been placed in the crushing chamber, and the lid has been clicked shut and not reopened.

**NO PEEKING!**

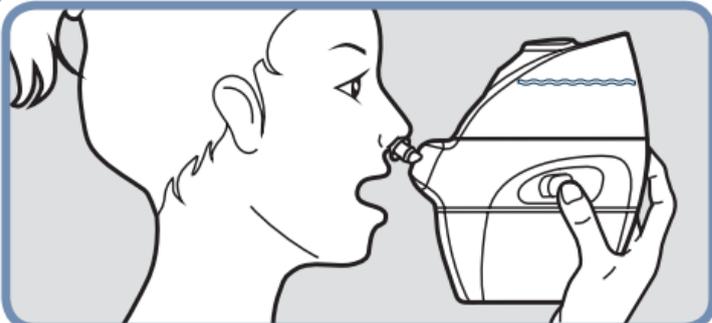
If you reopen the lid after loading a SaltPod, the device will not operate!

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
Correct posture while using the Naväge Nose Cleaner.

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
Correct position of nose pillows. Press firmly so there are no gaps.

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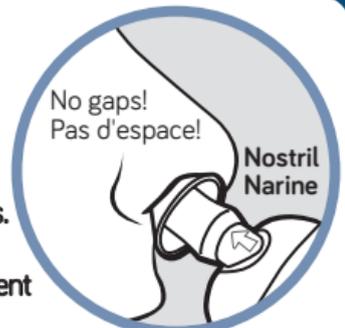1. Install batteries and assemble. See Owner's Manual, page 3.
2. Fill the upper tank to the fill line with warm water.
3. Place a new, unused SaltPod capsule in the crushing chamber, foil-side down.
4. Close the lid firmly until you hear it click shut and do not reopen it.
5. Wet nose pillows to make them slippery. Insert and center them in nostrils. Press firmly so there are no gaps.
6. **Relax...** Stand up straight with head erect. **Smile.** Breathe normally through mouth. Push power button in half-way and feel the suction for a moment or two, or push in all the way right from the start. Experiment and find what works best for you. Feel the saline flowing through your nose!
7. Cycle can last from 10 seconds to a minute depending on anatomy and congestion. Cycle is complete when upper tank is empty. Gently blow your nose.
8. Enjoy the moment. You've just taken the first step toward a lifetime of better breathing with Naväge.

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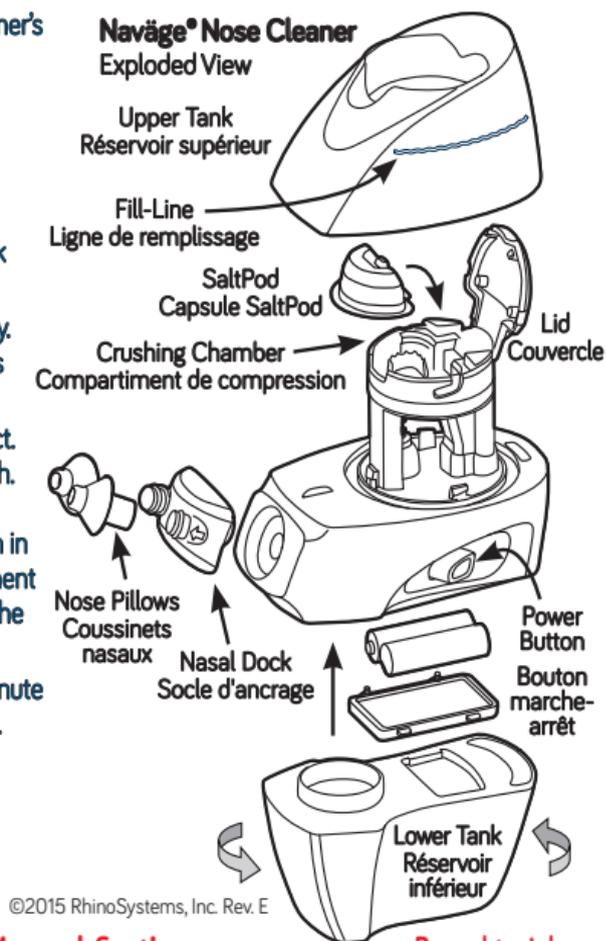**For best results, read Owner's Manual first!**

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